Purpose

The CASCADE wall handgrip is an ergonomically shaped handle, fully made of plastic. It is designed to be used in shower and bath areas in bathrooms or other care facilities.

The handgrip may be mounted vertically, horizontally or diagonally. The handgrip can be attached as a chain of grips as desired.

Care and Maintenance

The handgrip can be cleaned with a household plastic cleaner. Follow the instructions of the cleaning product.

Warranty

We offer a 2-year limited warranty against defects in workmanship and materials CASCADE wall handgrip.

NOTE:


Keep this user manual in case of future use.


For complaintsor questions, always provide the model and the Assembly Instructions no.

Package Contents

Pos.	Art. No.	Name	Pcs.
Α	6020501	Wall handgrip	1
В	6020504	Socket	1
С	6020506	Cone insert	1
D	6020508	Cap (2 pcs. extra)	4
Е	6020010	Carrier for perforated disk	2
F	6020012	Hole plastic plate	2
G	8505835	Cylinder screws M8x35 A2	2
Н	8520550	Universal screws 5.0x50	6
ı	8555008	Fischer wall anchor S8	6
J	8500008	Nut m8, DIN 985	2
K	8580006	Allen Key AF 6	1
	02.10.072	Assembly instructions Cascade	1

Photo / Contents

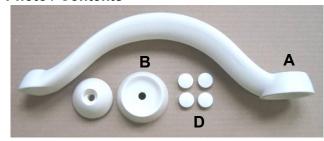

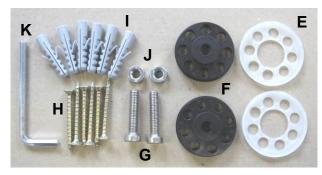

Assembly / Base Set (image similar)



Assembly / Extending the Handle



Instructions for Assembly

Before installing the wall handgrips, check to be sure there are no water pipes or electrical lines in the wall that could be damaged by mounting the handgrip.

REBOTEC is not liable for any damages resulting from mounting the handgrip on the wall.

When mounting the handgrip, the structural wall properties must be taken into account. If necessary, the screws and anchors may have to be replaced by another fastening system.

The distance between the hole-centres is 290 mm.




Check to be sure the handgrip is securely mounted after installation.

Trained professional

Handgrip Chains

For extending a chain of handgrips, the following parts are needed for installation:

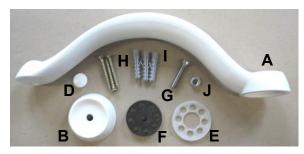

All other parts in the set will not be needed.

Attention

Place nut (J) with the integrated lock ring facing the wall.

Safety Instructions

- Always take the structural wall properties into account when mounting the handgrip.
- Do not use damaged or improperly mounted hand-grips

Areas of Application

Bathtub



Shower



We also supply

Walking aids

Shoulder crutches

Shower stools

Shower and commode chairs
Multifunction-Wheel Chairs Rollators

Patient lifting hoists Standing up aids

Your REBOTEC Team



REBOTEC® Rehabilitationsmittel GmbH PO Box Q541 Queen Victoria Building NSW 1230

rebotec.com.au info@rebotec.com.au



GB Assembly Instructions

MA No.: 02.10.072/2017-05

Wall Handgrip "CASCADE"

Art. No.: 601.05.00

 ϵ

