



Maintenance

Periodic safety-related controls are needed:

- Check for apparent damage:
(Cracks, fractures, loose parts).
- All screwed connections for tightness
(Especially the castor wheels and the frame).
- Function of the armrests and locking pins.
- Function of the footrests
(Replace defective footrests).
- Rolling ability, check function of the wheels
(Replace defective wheels).
- Check accessories for defects.



Repair

- If you have questions concerning maintenance and repairs to the chair, then contact an authorized dealer or medical equipment supply outlet.
- When making repairs, use only original spare parts from REBOTEC. Observe all advisories concerning guarantee and warranty.
- Never make any unauthorised modifications or conversions to the chair. These will only adversely affect the safety and function of the chair.
In such cases REBOTEC assumes no liability!
- Follow all the instructions when ordering spare parts, re-use and product lifetime of the chair.
- After a complete maintenance or repair of the chair is done, we recommend a complete cleaning for assuring hygiene. Follow the cleaning instructions.



Here you will find information on maintenance and forms for documentation.

You can order the User Manual and the maintenance report or see via our website at www.rebotec.com.au It is possible to print out the information and to read it as needed.