

Technical Specifications (Technical specifications subject to change without notice!)

Dimensions		Dijon	Brest	Lyon
Seat height (variable)	cm	42 - 57	42 - 57	42 - 57
Seat height (fixed)	cm	50.0	50.0	50.0
Height with armrest	cm	-----	61 - 76	-----
Height with armrest	cm	-----	-----	83 - 98
Seat size (LxB)	cm	46 x 40	46 x 40	46 x 40
Area required	cm	45 x 51	45 x 59	51 x 59
Seat height adjustment		7 times/ea. 25 mm	7 times/ea. 25 mm	7 times/ea. 25 mm
Weight	kg	3.0	3.8	5.0
Weight (patient)	kg	max. 130	max. 130	max. 130

Spare Parts

Article No.	Name	Remark / Colour
463.90.20	Armrests (incl. clips)	Plastic, white
463.90.30	Backrests (incl. clips)	Plastic, white
463.90.40	Sleeve nut	Plastic, white
201.90.28	Clips for legs (height adjustment)	Round, white
463.90.60	Clips for attaching the armrests and backrests	Square, red
442.00.25	Plugs for leg tubes	Plastic, black
456.00.25	Leg tips	Plastic, white

Accessories

284.00.30	Travel pouch for commode aid	Textile fabric, black
301.00	Toilet basin, grey	Plastic
309.90	Basin receptacle	Plastic, white
440.10.62	PUR seat cushion	black

Reuse

The product is intended for reuse. The reuse (life) depends on the product state. Before the re-use of the device maintenance must be carried out and the appropriate instructions to security, care and maintenance must be observed.

This review shall exclude a possible improper use caused by obvious or invisible damage. Please refer to the Medical Device Directive (MDD) and the regulation for operator (MPBetreibV).

Life

The expected life of the products listed in this manual when used in accordance with safety instructions, maintenance intervals and care instructions is **3 years**.

Beyond this period, the product can be set used only in a safe condition. Maintenance should be done at least every six months. Specific requirements of the product, due to the use of the patient, causing shorter maintenance intervals.

Warranty

The products listed in this manual are provided with a warranty of **24 months**. Wear parts and damages of the products due to improper use are excluded.

Advisory!

You can see this User Instructions at our website at www.rebotec.de.

It is possible to print out the information and to read it as needed.



User Instructions



REBOTEC

REBOTEC® Rehabilitationsmittel GmbH

PO Box Q541
Queen Victoria Building
NSW 1230

rebotec.com.au
info@rebotec.com.au

Serial No. / Product



Commode Aids

**Toilet Frame
Brest
362.00.20**



**Commode Chair
Lyon
362.00.30**



BA No.: 02.08.077/2016-12

Intended Use / User Advisory

The commode aids are to be used only in interior household and in-patient areas on even, solid surfaces and for using toilets. It serves to assist the physically impaired by increasing independence when using a toilet. Despite optimum sitting comfort, the chair is not suited for long term sitting.

Advisory!

Always keep the user manual in case of future use.


The identical serial number in the User Instructions is also printed on the nameplate of the product. For complaints or questions, always provide the model no., serial no. and the User Instructions BA No.

Product Advisories

The tubular frame is made of aluminium tubing. The seat height is adjustable (to 7 positions) using the telescoping aluminium tube. A non-adjustable version for seat height is also available as an option (single-piece tube). The leg tips for the stool legs are made of a slip-resistant plastic. The commode aid can be completely assembled or disassembled with the clip-system and thus offers easy mobility for location changes.



First-time Use

The commode aid is not delivered ready for use. It first has to be assembled before using it the first time. Using the packing list, check to be sure all parts are at hand before starting the assembly. The parts can vary depending on the commode aid model!

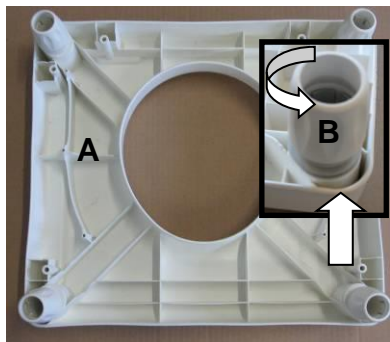


Model Dijon

- Place seat plate (A) on an even surface.
- Loosen sleeve nut (B) by left-hand turning (counter-clockwise).
- Insert tube (C) into the clamping element (D) until it strikes the end.
- Rotate the tube so that clip (F) is positioned outwards. Be sure all legs are installed in the same way!
- Tighten sleeve nut (B) by turning it to the right until firm.
- Install the other three legs in the same exact way.
- Pull out clip (F) completely and adjust the seat height by pulling out the inner tube.
- Insert clip (F) inside the borehole in the tube.
Warning! Push the clip all the way in! You will know when the clip is all the way inside the aluminium tube.
- **Adjust all 4 legs to the same height!**
- After assembly check the commode aid for stability!

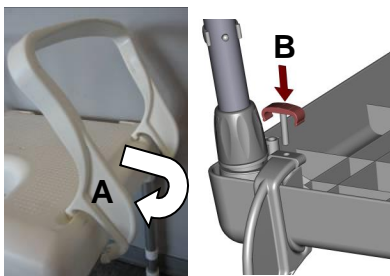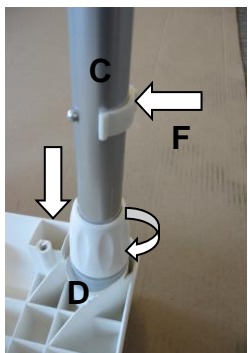

Model Brest

- Assembly same as model Dijon.
- Clamp the armrests (A) firmly onto the seat plate. Be sure that the clamping unit snaps on below the seat plate.
- Pull the armrest supports slightly apart until the boreholes line up and fasten with clip (B).
- **Attention! Press the clip all the way in!**

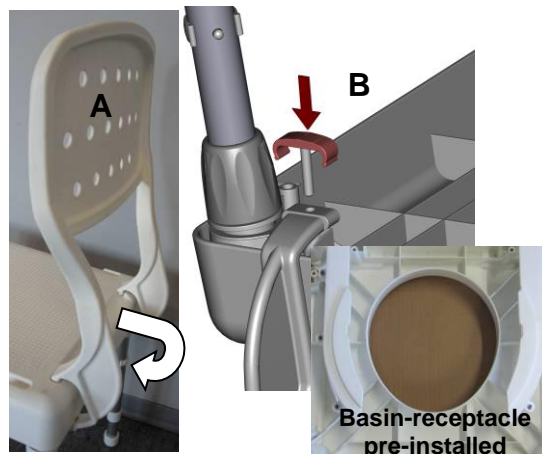

Model Lyon (with basin receptacle)

- Assembly same as model Dijon
- Armrest assembly same as with Model Brest.
- Clamp the backrest (A) firmly onto the seat plate. Be sure that the clamping unit snaps on below the seat plate.
- Attach backrest using clip (B).
- **Attention! Press the clip all the way in!**

Cleaning and Disinfection

The chair can be cleaned with a commercially available household cleaning agent. Do not use an aggressive or foaming type of cleaner. For disinfecting the surface, use a commercially available disinfectant (be sure to follow the instructions). Then wipe away with clean water and wipe dry. Inadequate or negligent care and maintenance leads to limitations or annulment of the warranty and product liability by the Manufacturer.

Maintenance

Regular maintenance and care of the commode aid assures long-term reliable use. For safety reasons and for preventing accidents whose cause may be due to undetected wear or damage resulting from improper use, be sure to have the commode aid checked at least every six months by an authorised dealer.



Safety Advisories

- **Do not exceed the stool's specified maximum allowable weight!**
- Before each use, carefully check the stool over for damage. If damage is detected, then do not use the stool!
- **Danger of tipping!**
- The following points must be checked before using:
 - ✓ Check the commode aid for stability (after assembly and before use).
 - ✓ Check seat plate for damage.
 - ✓ Check height-adjustment clips for firm mounting.
 - ✓ Depending on the model, check clip locking of the armrests and backrests.
- Do not carry the commode aid while the patient is on it.
- Use the commode aid only inside and on solid, even surfaces.
- Depending on the model, be sure that the armrests and backrest are used only when they are fully and properly installed.
- Do not lean too far out beyond the commode stool.