



REBOTEC® Rehabilitationsmittel GmbH
DE-49610 Quakenbrück, Artlandstr. 57-59
Telephone: +49(0)5431/9416-0
Telefax: +49(0)5431/9416-66
<http://www.rebotec.de>
e-mail: info@rebotec.de

Saddle seat unit for Reverse Walker Pluto

Mounting kit size 1 / Art.No.: 287.20.10

Mounting kit size 2 / Art.-Nr.: 287.20.20

Mounting




Accessories:
287.00.38 Back cushion



Adjusting the saddle seat height

If the saddle seat unit (weight ca. 1,6/1,8 kg) is mounted to the walker, the saddle seat height can be adjusted continuously (from 430/510 mm to 510/630 mm).

To adjust the saddle seat height, loosen the quick release on the saddle pickup tube. Adjust the seat to the needed height (by pulling the seat out).

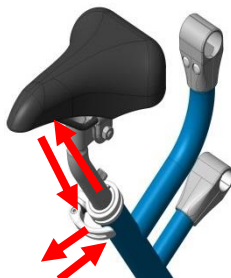

Align the saddle seat parallel to the frame and clamp the seat tube with the quick release firmly. **Attention! Max. setting height when the end of the pipe is visible in the slot.**